

Procedure Masks



Heating Option:

- Place mask(s) in heating unit. Ensure adequate air space between masks so all masks reach proper temperature.
- Masks should not contact metal surfaces of heating unit. This will degrade material and deteriorate fit.
- **Heat for minimum of 30 minutes at 70°C/158°F.**



DETAILS & WARNINGS

- Reuse masks to preserve supply.
- Do not attempt to reprocess visibly soiled masks (discard).
- Avoid using alcohol, bleach, radioactive gamma rays, lysol.
- Masks should not be shared among healthcare providers.

Hold Option:

- Place mask in clean brown paper bag or box.
- Write name and date on paper bag/box.
- Store at room temperature for at least 3 days in location that allows droplets to fully dry.