


SHIELDS

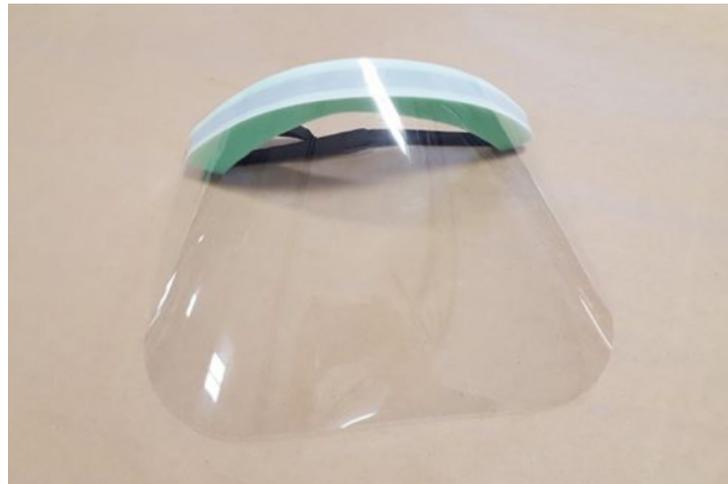

Cleaning:

- Wipe gently with 70% Isopropyl Alcohol.
- **Allow to completely dry.**

Hold Option:

- Place mask in clean brown paper bag or box.
- Write name and date on paper bag/box.
- Store at room temperature for at least 3 days in location that allows droplets to fully dry.



DETAILS & WARNINGS

- Reuse shields to preserve supply.
- Do not attempt to reprocess visibly soiled shields (discard).
- Avoid using Cavicide, bleach, radioactive gamma rays, lysol.
- Shields should not be shared among healthcare providers.